Smart Ring

User Guide & Safety Information

Begin your experience by reviewing this guide before using your Velia Smart Ring, charger, or app.
Proper setup ensures smooth performance and safety.

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User instructions

Welcome to Velia

To ensure the best experience, please take a moment to review this guide before using your ring, charger, or mobile app. Proper setup and care help prevent damage, malfunction, or injury.

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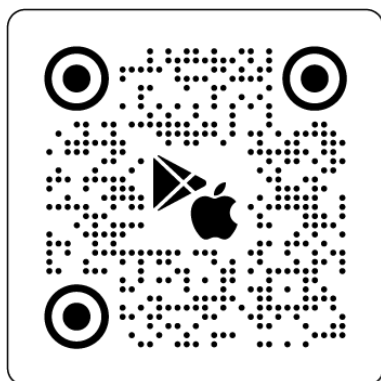
Let's Get Started

1


Download the Velia app from Apple App Store or Google Play.

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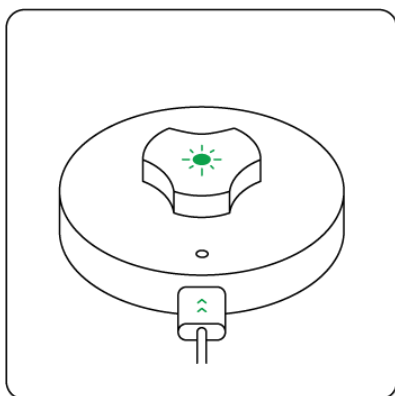2

Connect the charging station to a power source using USB-C cable.

The LED on the top of the charging station will glow a steady green when it's ready.

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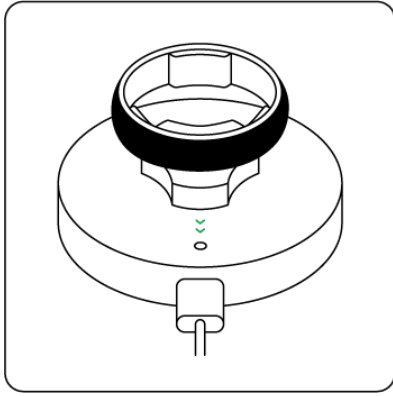
3

Place the ring on the charger, aligning the Velia logo inside the ring with the dot on the base of the charger.

A green pulse from the ring confirms it's charging correctly.

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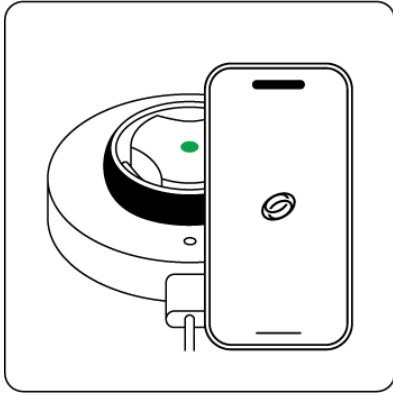
4


Open the Velia app and follow the steps to create your account and pair your ring.

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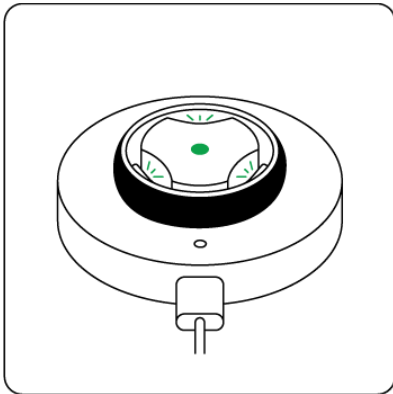5


When the LEDs inside the ring glow steady green, your ring is fully charged and ready to wear.

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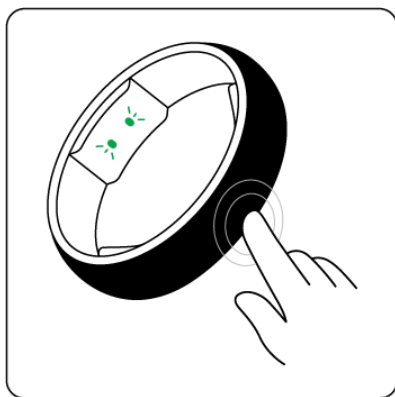6

Double tap to sync

Double tap anywhere on your ring and within seconds, it begins syncing your data.

You can reconnect anytime with another double tap.

While your ring is charging, it connects to the app automatically, keeping everything in sync while you power up.

A ring firmware update may be required. If so, you'll be prompted to install it from the Velia app.

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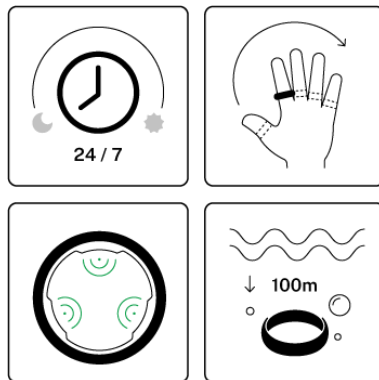
A Brief Overview

1

Velia is designed for all-day comfort and can be worn on any finger—just choose what feels right for you.

With 360° sensor technology and water resistance up to 100 meters, it delivers accurate insights no matter how it sits on your finger or where your day takes you.

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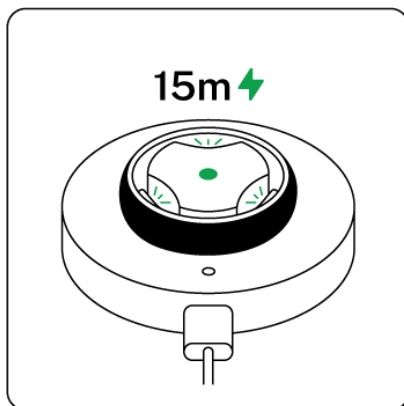
2

Just 15 minutes of charge a day keeps you in sync, so you never miss a beat.

Full charging takes 20-30 minutes, and battery levels are always visible in the app.

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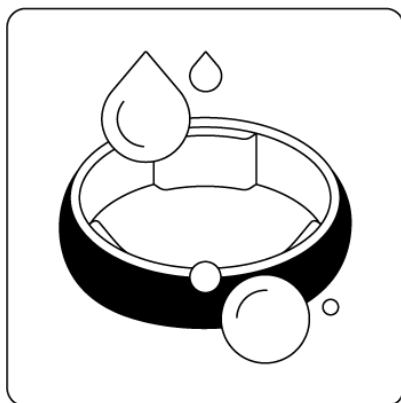
3

Clean your ring gently using a soft cloth with mild soap and water.

Avoid harsh chemicals or abrasive materials, and always dry the ring thoroughly after bathing, swimming, or intense activity.

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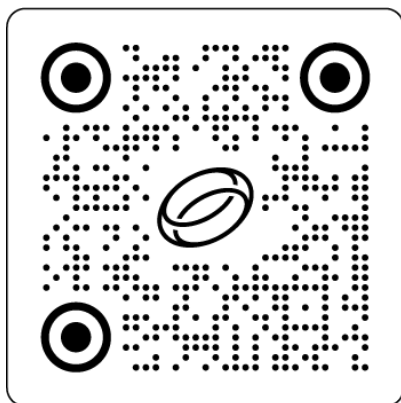
4

Have a question?

Visit our [FAQ](#) or reach out to our support team anytime at [veliaring.com](https://www.veliaring.com).

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Using Your Velia Smart Ring

Begin by following the [Let's Get Started](#) section to charge and pair your ring. For the best experience, keep the Velia app up to date to access the latest features and improvements.

Wearing

Your Velia Smart Ring should fit snugly but comfortably on any finger—secure without being restrictive.

⚠ Remove the ring before engaging in high-impact sports, using power tools, or any activity that may cause sudden pressure or abrasion.

Temperature & Environment

Your ring performs best between 10°C and 45°C (50°F to 113°F).

⚠ Avoid extreme temperatures, prolonged sun exposure, and storing the ring in hot environments, such as inside a vehicle. Leaving the ring in an extremely high temperature surrounding environment can damage the battery inside the ring and result in an explosion or the leakage of flammable liquid or gas.

⚠ Using the ring in extremely low air pressure conditions, such as those found at high altitudes, may cause it to explode or leak flammable liquid or gas.

Travel

Velia is travel-ready and safe to wear through airport security and during air travel.

While in flight, you may continue wearing the ring; however, airplane mode may limit real-time data syncing.

Sleep

Wearing your Velia Smart Ring overnight allows it to track your recovery, heart rate, and sleep stages. For the most accurate insights, wear the ring consistently during sleep.

Heart Rate & Activity

Velia monitors your heart rate, movement, and energy output throughout the day. The longer you wear your ring, the more tailored your insights become.

Wear & Care

Velia is designed for durability, but care is essential for long-term performance and comfort.

- Your ring may be worn during showers, baths, swimming, and snorkeling.
- Finger size may fluctuate due to temperature, activity, or hydration. If the ring becomes difficult to remove:
 - Raise your hand above your heart
 - Apply cold water and mild soap, then gently twist the ring off

⚠ If the ring causes pain or cannot be removed, seek medical attention immediately.

⚠ Avoid:

- Wearing the ring during strength training, shoveling, or while handling heavy metal or stone objects

- Wearing multiple rings in close contact to prevent surface scratching
- Prolonged exposure to heat or direct sunlight
- Contact between the ring and battery terminals or other high-conductivity surfaces

⚠ To reduce the risk of electrical shock, do not handle batteries while wearing your ring. As with any metal ring, there is a potential for short circuiting in rare conditions.

Support & Help

If you need help getting started or encounter any issues, visit our [FAQ](#) for frequently asked questions. If you can't find what you're looking for, reach out via [Contact Us](#) and our team will be happy to assist.

⚠ Safety Notice

Velia products are not medical devices and are not intended to diagnose, treat, cure, or prevent any medical condition. Insights are provided for informational and wellness purposes only and should not replace the advice of a healthcare professional. Always consult a medical provider before making health-related changes.

⚠ Do not ignore symptoms or delay seeking professional care based on Velia insights. If you experience persistent discomfort, fatigue, or unexpected physical changes, stop using the ring and consult a physician.

⚠ If you notice skin redness or irritation, remove the ring immediately. If symptoms persist after a few days, consult a dermatologist.

⚠ Do not wear the Velia ring during medical imaging such as MRI, CT scans, or X-rays. The ring contains metal and a rechargeable battery, which may interfere with medical equipment and post a risk of injury or device damage.

⚠ Take care to ensure your Velia Smart Ring does not catch on fixed objects during movement.

⚠ This product is not a toy. Do not place the ring in your mouth and keep it out of reach of children. If swallowed, seek emergency medical assistance immediately.

Terms of Use

The use of our products and services implies acceptance of our Terms of Use, as set forth in this document, as well as:

- Our Terms of Service, available at <https://veliaring.com/policies/terms-of-service>
- Our Privacy Policy, available at <https://veliaring.com/policies/privacy-policy>

⚠ These terms may be updated periodically without notice. Please refer to the latest versions on our website.

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Product specifications

Ring Materials

- **Outer surface:** Titanium with PVD coating
- **Inner surface:** Non-allergenic polycarbonate (PC) and polymethyl methacrylate (PMMA)

Data Storage

- Stores up to 5 days of activity and biometric data, depending on usage.
- Daily sync with the Velia app is recommended to ensure data is not lost.

Use of Personal Data from Sensors

- This product uses PPG (Photoplethysmography), accelerometer, temperature sensors to collect data that may include user information. Such data is used to derive health and activity insights including but not limited to motion, steps, heart rate, breathe quality and skin temperature.
- By using this device, you acknowledge and agree that such data will be processed in accordance with our privacy policy, and solely for the purpose of improving your experience and providing personalized feedback. We do not collect voice, GPS location, or microphone data.
- For more information on how your data is used and protected, please refer to the [\[Privacy Policy\]](#).

Battery

- Rechargeable lithium polymer (LiPo) battery
- Capacity: 3.4–6.0 mAh depending on ring size
- Battery is non-replaceable
- Typical battery life: 1–3 days depending on features enabled and usage behavior
- Charging method: Wireless charging via Velia charger
- Charging time: 20–30 minutes depending on the starting charge level

LEDs and Sensors

- Uses low-power light-emitting diodes (LEDs)
- Safe for users and those nearby

Connectivity

- This product uses short-range wireless communication technology (Bluetooth) to connect with your phone. Wireless connectivity is activated by default when the product leaves the factory. This setting enables seamless connection between your device and phone.
- Note: Your phone's location services may be required during wireless scanning, as mandated by your operating system's policies. However, we do not collect or store any GPS data.
- Wireless communication: 2402–2480 MHz @ 0.251 mW (-6 dBm)
- Encrypted communication between the ring and Velia app
- Firmware updates are delivered via the Velia app

App Compatibility

- Velia app available on the App Store and Google Play
- Compatibility details available at: veliaring.com/pages/faq

Water Resistance

- Ring is water resistant up to 100 meters / 328 feet (IPX8)
- Operating temperature range: 10°C to 45°C (50°F to 113°F)

Charger Specifications and Usage

Usage Guidelines

- Charger is not water resistant; intended for indoor use only
- Do not use or store in humid environments (e.g., bathrooms)
- Recommended charging temperature: 18–25°C (64–77°F)
- Do not wear the ring while charging

Technical Details

- Input: USB Type-C connector, nominal 5 V
- Must be powered by an external 5 V DC supply compliant with IEC 62368-1
- Power input must not exceed 2.5 W after one second of connection

Disposal and Compliance

- Dispose of the ring, which contains an internal battery, in accordance with local regulations. Special handling may be required.
- Never dispose of the ring into fire or a hot oven, or mechanically crush or cut the battery inside the ring.
- The charger is classified as electronic waste—follow local e-waste recycling guidelines
- To perform a factory reset prior to disposal, visit: veliaring.com/pages/faq

Warranty disclaimer

The Services are provided "as is" and "as available" without warranties of any kind. We do not warrant that the Services will be uninterrupted, error-free, or compatible with all devices.

Velia products are not certified medical devices and must not be relied upon for medical diagnosis or treatment. Always consult a qualified healthcare provider for health concerns.

Limited Warranty

We warrant the Velia ring and charger against material defects in workmanship and materials under normal use for the following periods from the date of original purchase:

- **Switzerland & EU:** Two (2) years minimum (per statutory law);
- **United States & Canada:** One (1) year;
- **United Kingdom:** Two (2) years (per UK consumer law);
- **Other jurisdictions:** As required by local law.

If a product is replaced or repaired under warranty, the replacement or repaired unit is covered for the greater of 90 days or the remainder of the original warranty period.

This limited warranty excludes damage from misuse, accidents, unauthorized modifications, or environmental conditions (e.g., extreme temperatures, humidity, exposure to liquids beyond rated specifications). It also does not cover normal wear and tear, including but not limited to battery degradation over time, cosmetic damage, or performance decline due to regular usage.

For clarity, "battery degradation" refers to the natural and expected decline in battery capacity over time due to charging cycles, environmental exposure, and usage patterns. This form of wear is not considered a defect. Only in cases of abnormal or excessive degradation during the warranty period, as determined by Velia at its sole discretion, may warranty claims be reviewed.

This limited warranty applies only to the original purchaser of the product and is non-transferable. All warranty claims are subject to confirmation of the reported defect by Velia. We reserve the right to request diagnostic information, photos, or return of the product for inspection. Valid proof of purchase from an authorized sales channel is required to process any warranty claim.

Products obtained through unauthorized resellers, third-party marketplaces not listed on our website, or second-hand sales are not covered by this warranty. We reserve the right to request proof of purchase to confirm eligibility.

Warranty remedies are limited to repair, replacement, or refund at our discretion. Any replacement product may be new or refurbished and will be functionally equivalent to the original product. Replaced products or parts become the property of Velia.

Shipping costs associated with returning a product for warranty service are your responsibility, unless otherwise required by applicable law. In addition, original shipping and handling charges paid at the time of purchase are non-refundable under any circumstances, including in the case of valid warranty claims.

To initiate a warranty claim, you must contact our customer support team through our official website or via support@veliaring.com. We do not process warranty requests submitted through unauthorized channels.

Limitation of liability

To the maximum extent permitted by law, Senbiosys SA shall not be liable for:

- Loss of profits or revenues;
- Loss or corruption of data;
- Loss of goodwill or anticipated savings;
- Personal or emotional distress;
- Any indirect, incidental, punitive, special, or consequential damages.

Our total liability shall not exceed USD 100.

Nothing in these Terms excludes liability that cannot be lawfully limited, including death, personal injury, or gross negligence.

Intellectual property rights

All content, software, algorithms, and designs associated with the Services, including the Velia™ name and logo, are the intellectual property of Senbiosys SA or its licensors. You may not use, reproduce, distribute, or alter any of our assets without explicit authorization.

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2012/19/EU

This product is compliant with Directive 2012/19/EU. Please dispose used electrical and electronic equipment according to national laws and regulations with electrical waste (WEEE). It should not be mixed with general household waste.

2014/53/EU

In order to satisfy the essential requirements of 2014/53/EU Radio Equipment Directive (RED), the product is compliant with the following standards.

Essential requirement

Article 3.1 (a) Safety

Article 3.1 (a) Health

Article 3.1 (b) EMC

Article 3.2 Radio

Article 3.3 Cyber security

Standards

EN IEC 62368-1:2020+A11:2020

EN 62479:2010

EN IEC 62311:2020

EN 55032:2015+A11:2020+A1:2020

EN 61000-3-3:2013+A1:2019+A2:2021

EN IEC 61000-3-2:2019+A1:2021+A2:2024

EN 55035:2017+A11:2020

EN 301 489-1 V2.2.3

EN 301 489-3 V2.3.2

EN 301 489-17 V3.3.1

EN 300 328 V2.2.2

EN 300 330 V2.1.1

EN 18031-2

2011/65/EU

This product is compliant with RoHS Directive 2011/65/EU and amendment directives (EU) 2015/863 on Lead, Cadmium, Mercury, Hexavalent Chromium, PBBs & PBDEs, Phthalates (DBP, BBP, DEHP, DIBP) content.

EU Authorised Representative

24hour-AR

Van Nelleweg 1, 3044 BC Rotterdam, The Netherlands
compliance@24hour-ar.com

France

United Kingdom

Hereby, Senbiosys SA declares that the radio equipment type Velia is in compliance with UKCA. The full text of the declaration of conformity is available at the following internet address:

[UK Declaration of Conformity \(DoC\)](#)

UK Authorised Representative

24hour-AR

15 Beaufort Court Admirals Way, Canary Wharf, London, United Kingdom E14 9XL
compliance@24hour-ar.com

USA

Senbiosys SA has not approved any changes or modifications to this device by the user. Any changes or modifications could void the user's authority to operate the equipment.

California Proposition 65

WARNING: This product may contain chemicals known to the State of California to cause birth defects or other reproductive harm.

FCC regulatory information

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

RF exposure safety

Velia Smart Ring complies with FCC radiation exposure limits set forth for an uncontrolled environment.

Velia Charger complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20 cm between the radiator and your body.

FCC ID: 2BKU5-SBDV0106	(Ring US6)
FCC ID: 2BKU5-SBDV0108	(Ring US8)
FCC ID: 2BKU5-SBDV0110	(Ring US10)
FCC ID: 2BKU5-SBDV0112	(Ring US12)
FCC ID: 2BKU5-SBDV02ME	(Charger US6 & US8)
FCC ID: 2BKU5-SBDV02LA	(Charger US10 & US12)

Canada

This Class B digital apparatus complies with Canadian ICES-003 (B).

Regulatory notices for Canada are available in this website: veliarings.com/declaration-of-conformity.

Innovation, Science and Economic Development Canada (ISED) regulatory information

This device complies with the Interference-Causing Equipment Standards and Innovation, Science and Economic Development Canada's license-exempt RSS standard(s). Operation is subject to the following two conditions:

1. This device may not cause interference; and
2. This device must accept any interference, including interference that may cause undesired operation of the device.

Radiation Exposure Statement

Velia Smart Ring complies with the Canada portable RF exposure limit set forth for an uncontrolled environment and are safe for intended operation as described in this manual.

Velia Charger complies with IC RSS-102 radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20 cm between the radiator and your body.

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